

U-Treat V4 Upgrade Installation Kit

Kit Part Number:
03-21-0031 = U-Treat Version V4

Installation Instructions

Document: TD-09-06-3042

Revision: B

If your upgrade kit was mailed to you, you should have received one USB flash drive containing the following files.

FOLDERS:

Backup

Default_AR010

Manual

INDIVIDUAL FILES:

Arnbcfg.txt


Notes.txt

USCAutoRun.cmd

If you received the download link via e-mail from a USC technician, you will need to supply the flash drive and download the files to that


DOWNLOADING FILES

STEP 1: The files are too large to send as an e-mail attachment. The e-mail will include a link. Select the link and the download will begin automatically. A internet window will open and the download progress will be shown in the lower left corner. When it is complete go to your download folder.

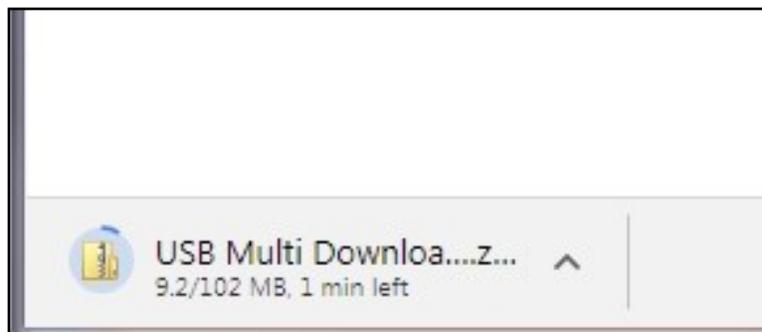


DOWNLOADING FILES

STEP 2: Open your download file. Double click the zip file, then double click on the folder. You should see the three folders and three individual files. Copy and paste the three folders and three individual files onto your blank USB drive.

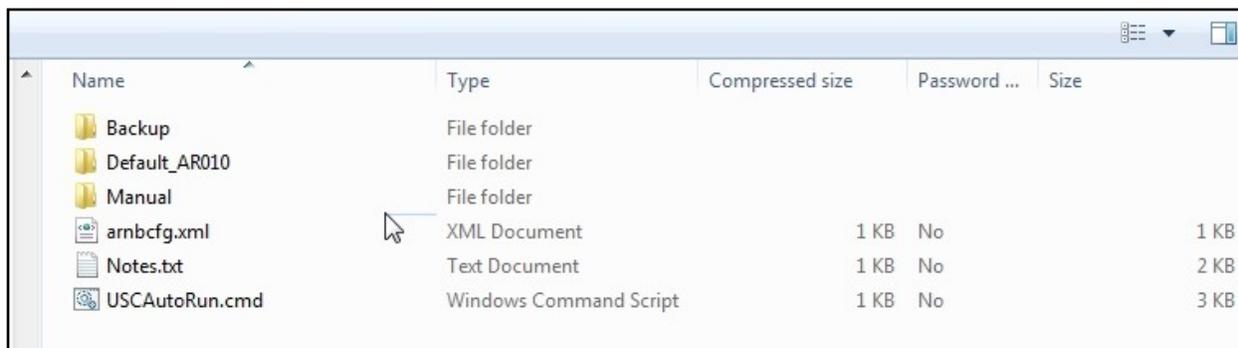

If double clicking does not unzip the file, right click on the file and select **Extract All...** from the dropdown menu. Press the extract button on the popup screen to unzip the files.

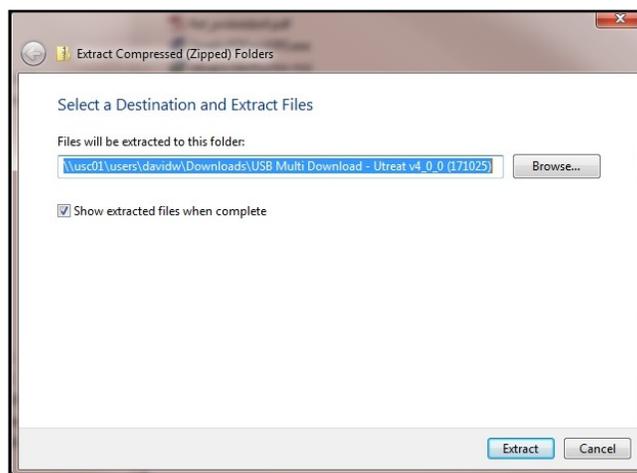

STEP 3: Copy the folders and files onto the USB flash drive and remove it from the desktop.

PRESERVING v4.0 SYSTEM DATA

NOTICE

Updating the program will erase all nine system logs and profiles. It is necessary to export the nine system profiles from the system for re-entry into the system after the upgrade process is complete.

Press the Utilities button at the bottom of the screen. Press the Import & Export button on the right. Insert a blank flash drive in one of the USB ports on the bottom of the panel. When the USB Status on the upper left corner reads Connected, you may begin downloading. Press each one of the Export buttons for Chemical Profiles through VFD Profiles one at a time. There will also be a warning above the module, Please do not "Exit" or cycle power. Exiting or shutting off the power will stop the process before it is complete. Remove the flash drive.

idle
Import & Export
U-Treat 4.0.00
10/11/2017 15:17:20

idle
User: OPERATOR

USB Status:
Connected.

Select the specific list you would like to export,
import or delete.

Import/Export Status:
Please do not "Exit" or cycle power.

List	Total Used Records	Status	Actions		
Alarm Log	158	Exporting 40	Export		Delete
Events Log	224		Export		Delete
Messages Log	0		Export		Delete
Job Reports	1		Export		Delete
Customer Profiles	0		Export	Import	Delete
Seed Profiles	1		Export	Import	Delete
Chemical Profiles	0		Export	Import	Delete
Chemical Recipes	0		Export	Import	Delete
Pump Profiles	1		Export	Import	Delete
Bin Profiles	1		Export	Import	Delete
Conveyors Profiles	4		Export	Import	Delete
Paths Profiles	1		Export	Import	Delete
VFD Profiles	3		Export	Import	Delete

Date & Time

Print Setup

E-mail Setup

Import & Export

About

Overview
Treater Main
Treater HOA
Bin Site Main
Bin Site HOA 1
Bin Site HOA 2
Utilities
Reports
Profile Editing Screens
Help
Alarms
Security
Setup & Options
Setpoints
TREAT

BACKUP PROGRAM DAT FILES

There have been changes to the profile editing screen structures in v4.1. This will require backing up the v4.0 DAT files to a new folder name prior to installing v4.1.

Step 1: Tapping the blank spot in the upper right corner of the Import & Export screen once will bring up the far right side of the keyboard icon. Tapping the spot a second time will show a little more of the icon as seen in the enlarged view below. Tapping it a third time will bring up the entire keyboard.


Tap the program button once to illuminate it, then a second time to bring up the program popup as well as the system folder shown at the bottom of the screen. Press it to bring up the Data (D): folder in windows.



BACKUP PROGRAM DAT FILES

Step 2: Double tap the **ARWin** folder. Tap the **usc** folder once. Press and hold it a second time until there is a circle around the touch point. Release from the screen and a drop down menu will appear. Tap **rename** and change the folder name to **usc_4.0_backup_(date)**.

As the v4.1 program is loading it will create a new **usc** folder with all of the correct DAT files.

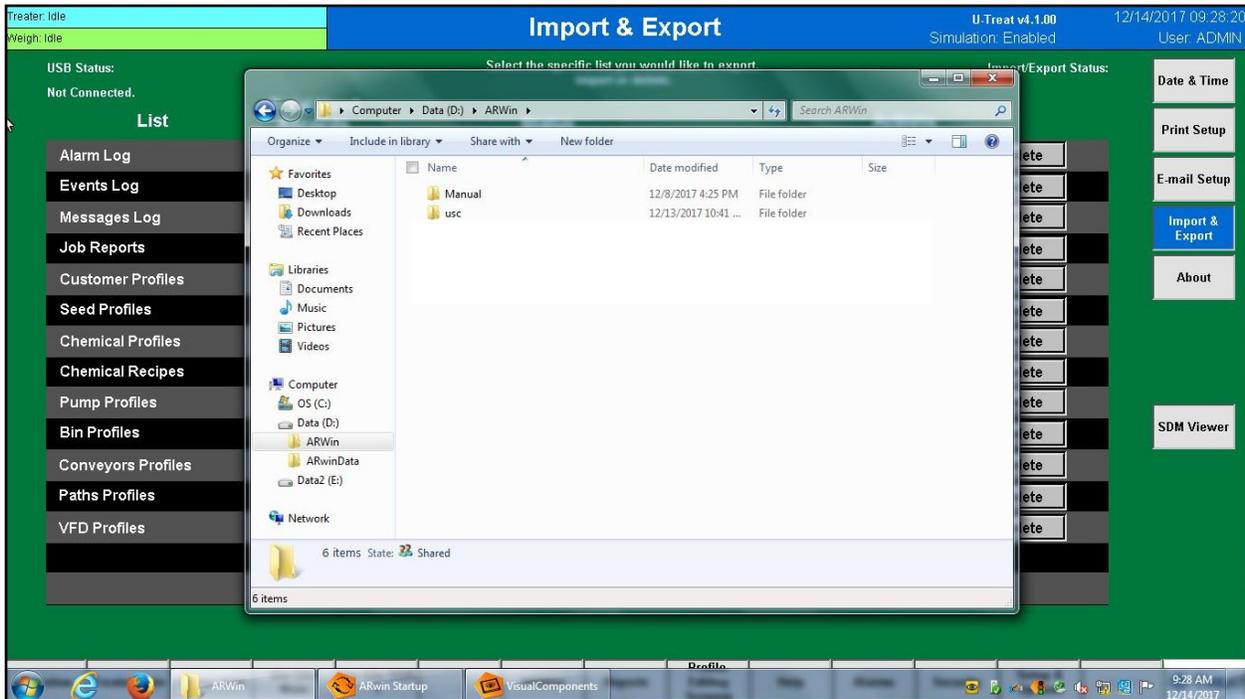


LOADING v4.1 PROGRAM

Step 1: Insert the flash drive with the program upgrade on it into one of the USB ports on the panel. The program will automatically begin to download. It will take approximately 10 minutes load.


LOADING v4.1 PROGRAM

Step 2: When the program is finished loading this is the first screen the operator will see after the program is finished loading. Remove the flash drive. After reading the User Acknowledgement statement, push the Acknowledge button at the bottom of the popup window to close the screen. While the system is booting up, the touch screen will display a timer bar at the bottom of the Start Up Screen. Once the timer bar reaches the end it will disappear and be replaced with a line of text that reads Press Screen to Continue. Press anywhere on the screen and It will advance to the Main screen.

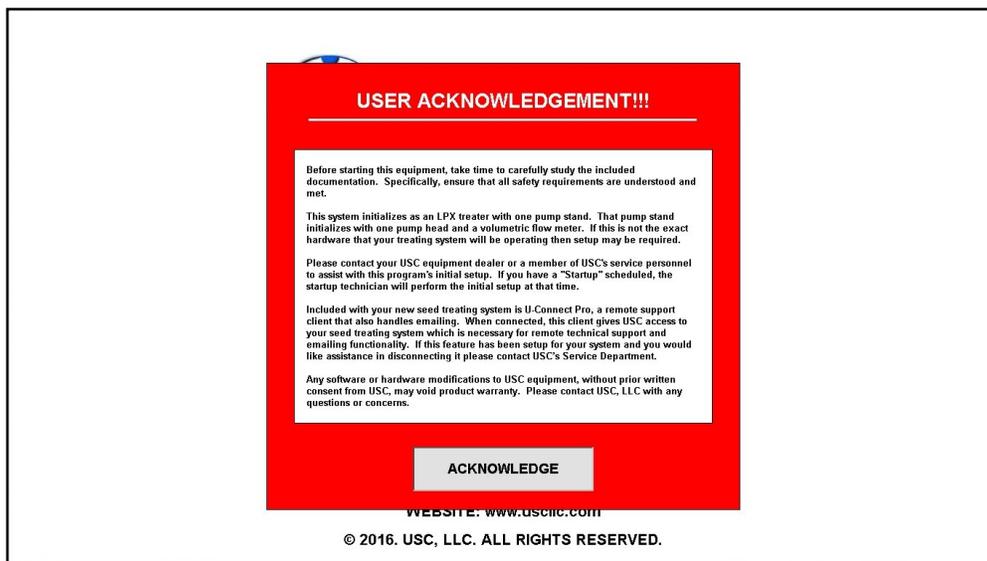


ESTABLISHING THE SYSTEM PARAMETERS

The first screen will be the main treater screen with a flashing emergency stop popup. The user name will also be set to DEFAULT. The system is alarming because none of the system parameters have been established. The following steps will take you through the process to set up the system to accommodate your treater and or bin site.

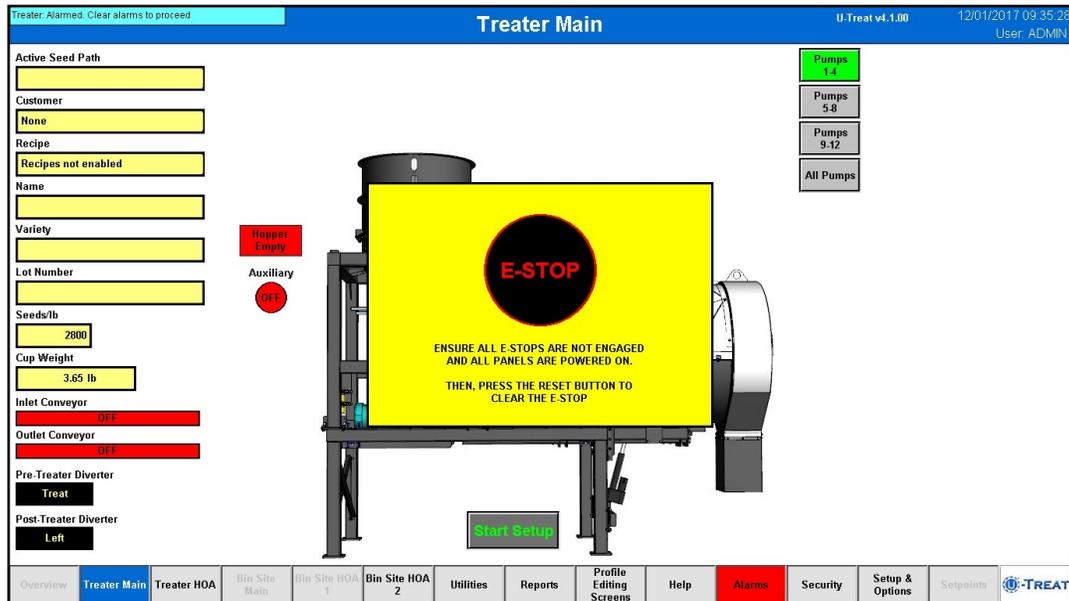


Step 1: Press the Security button at the bottom of the screen

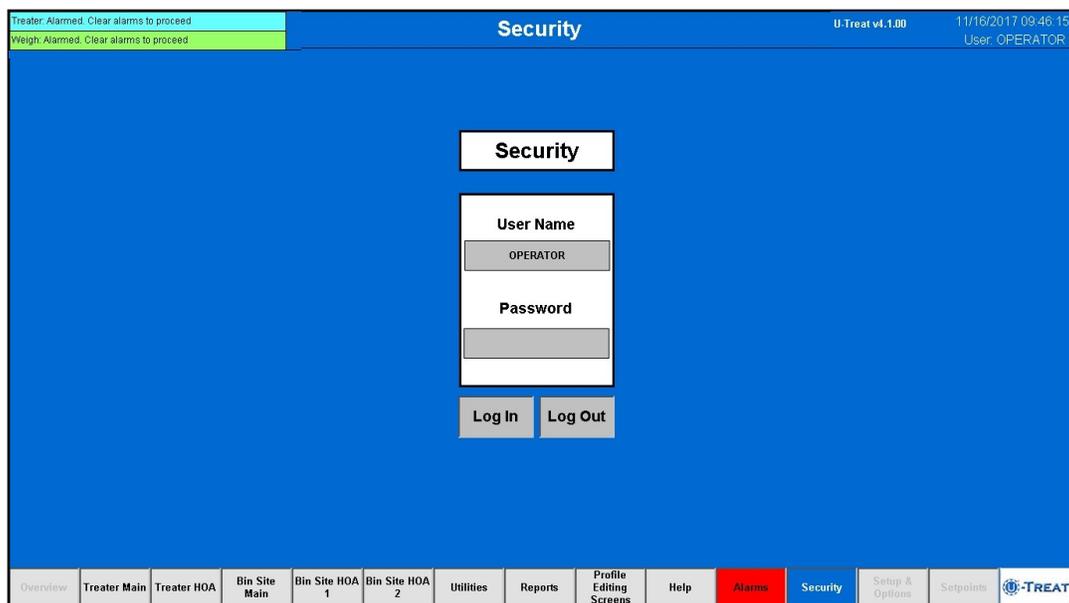

ESTABLISHING THE SYSTEM PARAMETERS

Step 1 (continued): Press the User Name button and a keyboard will popup with white background and grey text. This means it is set for lower case text. Press the shift key and it will change to dark grey background with white text for upper case. Type ADMIN in upper case text and press the key with the check mark to enter. Press the Password button and type SERVICE in upper case text and press the enter key.

Press the Login button and the User in the upper right hand corner will change from DEFAULT to ADMIN. The Setup & Options button in the lower right corner of the screen is now active.

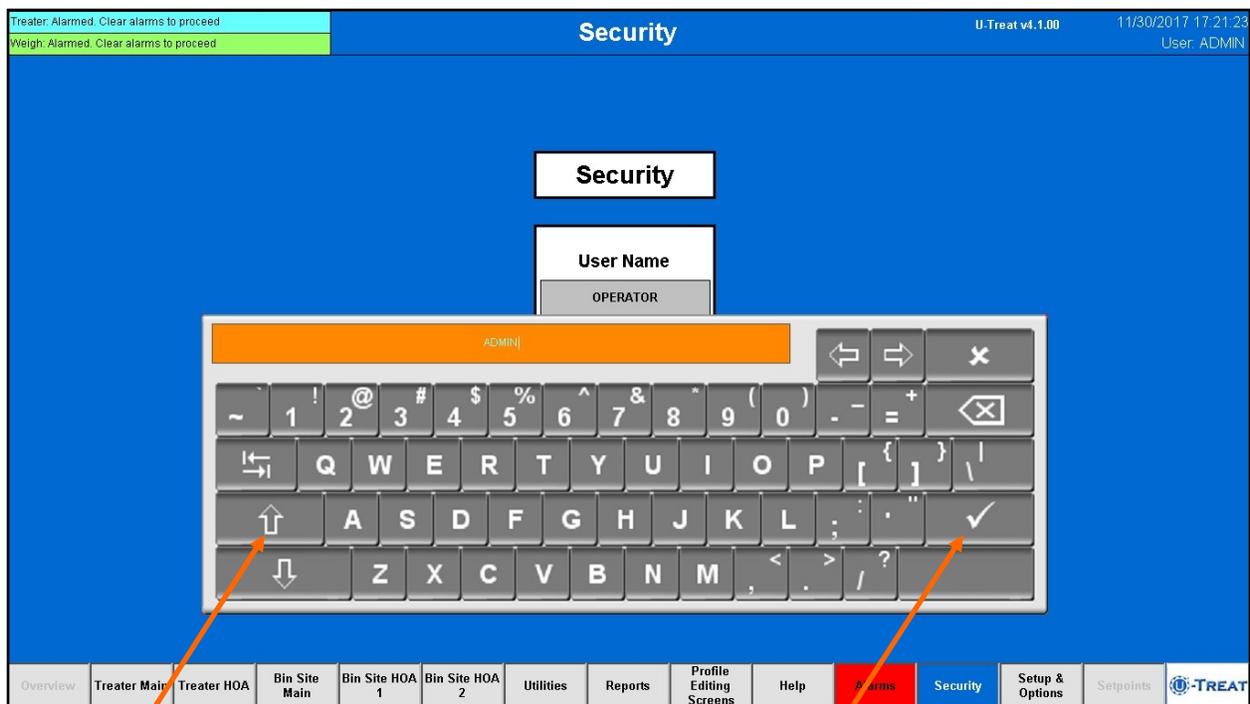

*Shift
Key*

*Enter
Key*

ESTABLISHING THE SYSTEM PARAMETERS

Step 2: Press the Setup & Options button at the bottom of the screen. This is where all of hardware parameters are established.


1. Press the Auto Detect button. This will establish communication from the new main control panel to the existing control panels in the system. Some of the boxes will become checked and some of the other parameters will be set. You will still need to add some of the more detailed information. Select the items on this screen to activate the parameters that apply to your existing hardware configuration. The treater options are on the top half and the bin site options at the bottom.

The example on page 27 shows an LPV2000 loss in weight treater with version 4 main control panel. Also, the drum tilt and treater batch mode are active as well as a pre-treater and post treater diverters. The bin site is also enabled with a Tri-Flo® weighing device, Rice Lake 1280 scale head and power link interconnectivity.


The screenshot displays the 'Setup & Options' interface. The top status bar shows 'Treater: Alarmed. Clear alarms to proceed', 'U-Treat v4.1.00', and the date/time '12/01/2017 08:33:03'. The user is identified as 'ADMIN'. The main area is divided into several sections: 'Treater Options' (Enabled), 'Configurable Treater Options', 'Treater Device Simulation', 'Treater Device Options', 'Pump Device Simulation', and 'Bin Site Options' (Disabled). The 'Bin Site Options' section is highlighted with a red box. On the right, a vertical menu contains buttons for 'Simulation Mode Disabled', 'E-mail Settings', 'Standard/Metric', 'Languages', 'Auto Detect', 'Configure Hardware', and 'Reset Hardware'. Three orange arrows point to these buttons, labeled with the numbers 1, 2, and 3. The bottom navigation bar includes buttons for 'Overview', 'Treater Main', 'Treater HOA', 'Bin Site Main', 'Bin Site HOA 1', 'Bin Site HOA 2', 'Utilities', 'Reports', 'Profile Editing Screens', 'Help', 'Alarms', 'Security', 'Setup & Options', 'Setpoints', and the U-TREAT logo.

ESTABLISHING THE SYSTEM PARAMETERS

Step 2: (Continued).

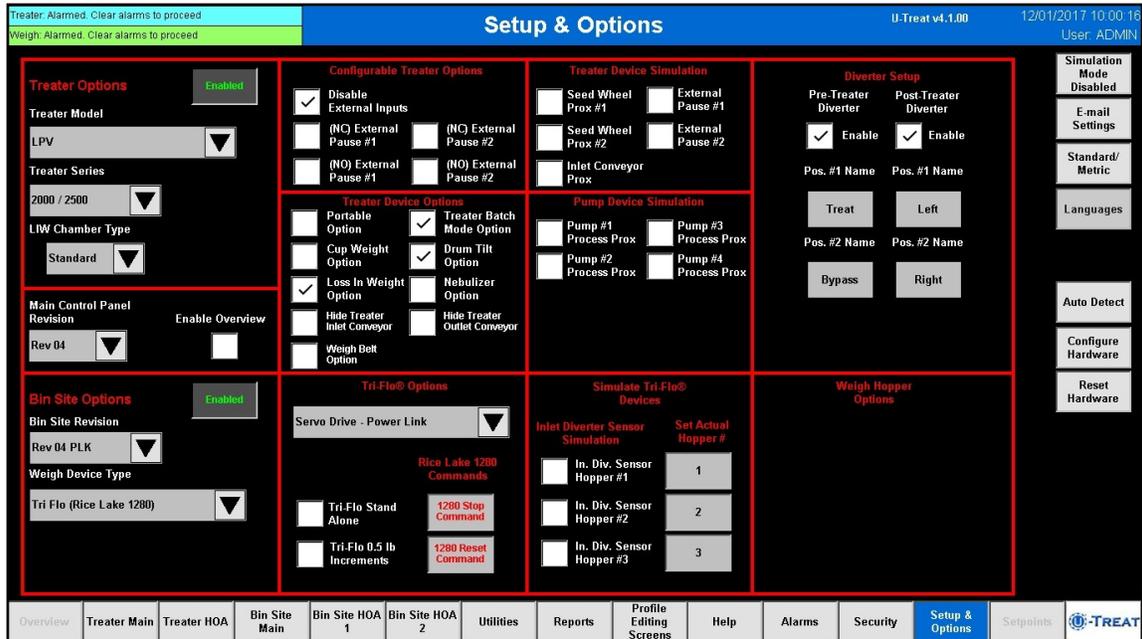


- Pressing this button allows the operator to switch between Standard or Metric units of measurement. When this button is pressed a popup window will appear. The box in the center indicates the active unit of measurement. The default is standard unit. Pressing the metric button to change to metric measurement.

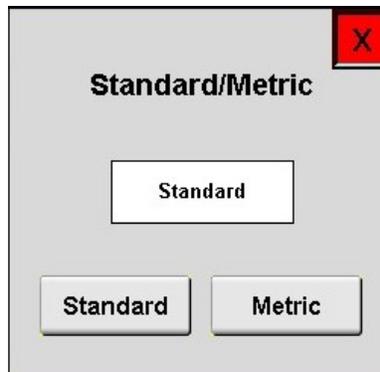

ESTABLISHING THE SYSTEM PARAMETERS

3. If you want to use the e-mail feature define the following e-mail settings.

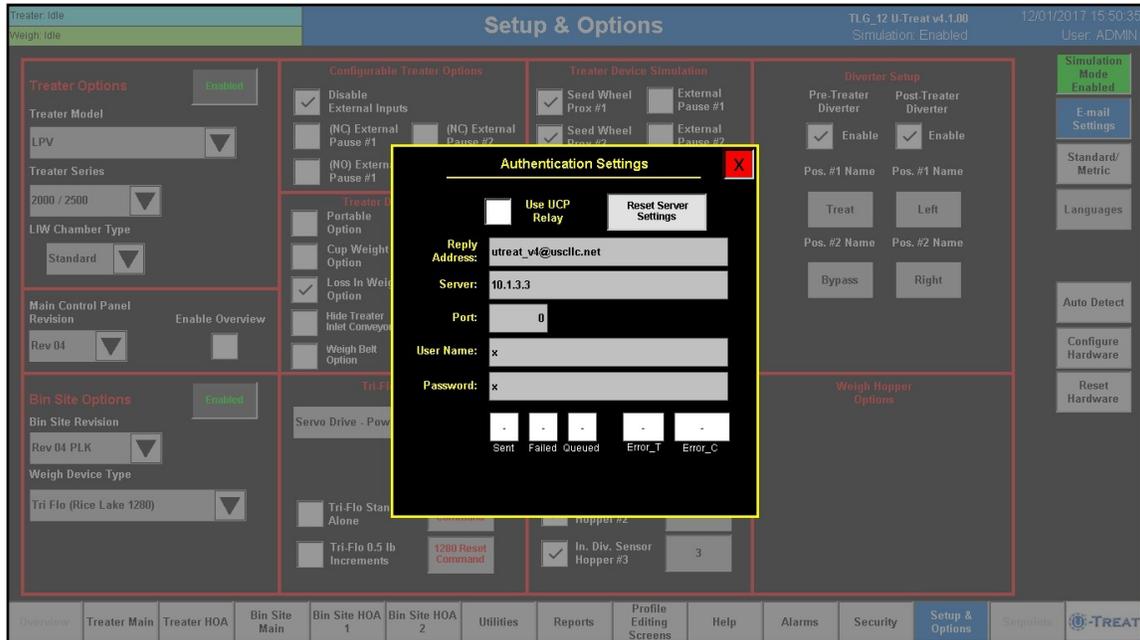

USE UCP RELAY: This setting is enabled if the customer is using U-Connect-Pro to relay the e-mail. If this option is not selected, the customer will need to provide their own mail relaying.

RESET SERVER SETTINGS: This button will reset the settings for the e-mail client.

Reply Address: This will be what e-mail address the e-mail is coming from. If there is a failure to deliver message, this e-mail will receive notification of the failure.
EXAMPLE: A misspelled e-mail destination.

Server: This will be the server that we will use to authenticate the e-mail delivery.

Port: This is the port number for the authentication server.

User Name: This is the username for the authentication server. This will be used to confirm that we have rights to use the authentication server.

Password: This is the password for the authentication server.

ESTABLISHING THE SYSTEM PARAMETERS

Step 3: Set the date and time. Press the Utilities button at the bottom of the screen, then the Date & Time button in the upper right corner. Select the top three boxes to set the year, month and day. Select the bottom three boxes to set the time. The system is based on a 24 hour clock. When keying in the hour, 2:00 P.M. is 14 hours as in the example below. If you wish to view the time on the screens in 12 hour display, check the box below the time setting. The display in the upper right hand corner will now show a 12 hour clock indicating A.M. or P.M. Press the Set Date & Time button in the center of the screen to save your entries.

Treater: Idle
Weigh: Idle

Date & Time

U-Treat v1.1.00 11/28/2017 02:28:50 PM
User: OPERATOR

Date & Time
Print Setup
E-mail Setup
Import & Export
About

Internal PLC Clock
11/28/2017 14:28:50

Date must be entered in the following format YYYY / MM / DD.

Year	Month	Day
2017	11	28

Set Date & Time

Time must be entered in 24-hour format. (23:59:00)

Hour	Minute	Second
14	28	50

Enable Title bar Date and Time 12h format display mode

Overview Treater Main Treater HOA Bin Site Main Bin Site HOA 1 Bin Site HOA 2 Utilities Reports Profile Editing Screens Help Alarms Security Setup & Options Setpoints TREAT

LOADING PRESERVED SYSTEM DATA

Step 5: Load the preserved data from the version 3 program into the v4.1 program.

1. Insert the flash drive into the panel. The USB Status in the upper left corner should read Connected. Import each one of the seven profiles back into the version 4 program.
2. As you begin to load the files back into the system a pair of buttons will appear underneath the Actions buttons (top). Importing the profiles copies them into the system. After all of the profiles have been imported, press the flashing green Align Data Tables button. This will place the files in the correct location. A popup timer will appear with two timer bars showing the alignment progress (bottom). Wait until it disappears before leaving the screen. This will synchronize the imported data with the updated system.

The screenshot shows the 'Import & Export' interface. At the top, it says 'U-Treat v4.1.00' and '12/13/2017 10:33:18'. The USB Status is 'Connected'. Below this is a table with columns: List, Total Used Records, Status, and Actions. The table contains the following data:

List	Total Used Records	Status	Actions
Alarm Log	247		Export Delete
Events Log	8708		Export Delete
Messages Log	8		Export Delete
Job Reports	68		Export Delete
Customer Profiles	1		Export Import Delete
Seed Profiles	2		Export Import Delete
Chemical Profiles	5		Export Import Delete
Chemical Recipes	1		Export Import Delete
Pump Profiles	5		Export Import Delete
Bin Profiles	2		Export Import Delete
Conveyors Profiles	4		Export Import Delete
Paths Profiles	1		Export Import Delete
VFD Profiles	4		Export Import Delete

At the bottom of the table area, there are two buttons: 'Align Data Tables' (green) and 'Cancel Data Table Align' (red).

The screenshot shows the same 'Import & Export' interface, but with a modal dialog box overlaid. The dialog box contains the following text:


Alignment of Data Tables In Progress ... Please Wait.

Aligning Bin And Seed Data Tables ...

Aligning Recipe And Chemical Data Tables ...

The dialog box also features two progress bars, one for each of the alignment tasks mentioned above.

NOTES:

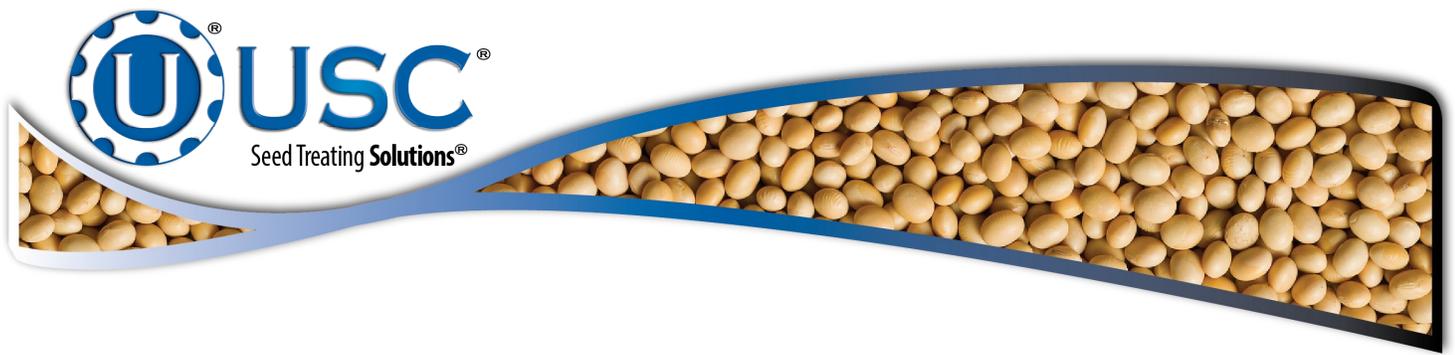

USC, LLC

2320 124th road

Sabetha, KS 66534

PHONE: (785) 431-7900

FAX: (785) 431-7950

EMAIL: sales-team@uscllc.com

WEB: www.uscllc.com
